

A Walk Around Tour of the HaloFogger®

Rear Handle

Status Indicator Light

High Precision Nozzle

Dry-mist of disinfectant is dispensed through nozzle.

Timer Dial

Turn the dial to set time for fogging treatment.

Fogging Time Chart can be found in the HaloFogger User Manual and on the rear panel of the Halo unit.

Start Button

Press and hold for 2 seconds to begin unit's countdown and the treatment process.

Disinfectant Fluid Level Indicator

Shows how much disinfectant is in the reservoir and when it is time to refill.

Reservoir Door

Push and lift open to expose funnel port for refilling.

Front Handle




Reservoir Door

Funnel Port



Large Wheels

For easy maneuvering.

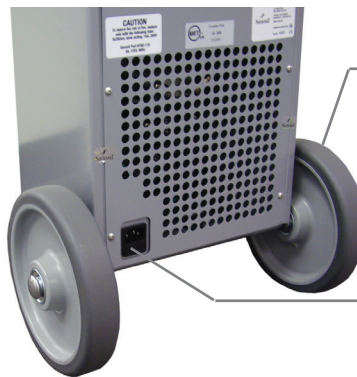

Power Cord Connection

For more information, please contact: